LBM<sup>®</sup>  
SLsolution™

Package insert and How-to-use guide

## Copan SL Solution - Presentation and guide to use of the product

See the explanation of the symbols at the bottom of this leaflet.

### INTENDED USE

Copan SL Solution is a treatment reagent for fluidification of specimens collected from the respiratory tract prior to subsequent microbiological analyses.

### SUMMARY AND PRINCIPLES

Copan SL Solution consists of a test tube with screw cap containing a ready-to-use solution and is indicated for fluidification of specimens collected from the respiratory tract, *Mycobacterium spp* excluded.

Copan SL Solution is packed so as to maintain a modified atmosphere that guarantees the stability and activity of the DTT reagent<sup>1,2</sup> until the time of use of the product.

### PRODUCT DESCRIPTION

Copan SL Solution is available in different configurations as indicated below and supplied in a labelled and closed test tube with screw cap filled with different volumes of reagent. The kit versions also include a sterile specimen transfer device, Pasteur pipette or sputum dipper. The .A codes are compatible with the automated device.

REF	Product description	Packaging
095CE	Copan SL Solution kit: - 20 12x80mm PP tapered-bottom test tubes with screw cap containing 1ml of reagent - 20 Pasteur pipettes	20 kits per package 10 x 20 kits per box
095CE.A	Copan SL Solution kit: - 20 12x80mm PP tapered-bottom test tubes with screw cap containing 1ml of reagent - 20 Pasteur pipettes	20 kits per package 10 x 20 kits per box
0E004N.A	Copan SL Solution kit: - 50 12x80mm PET round-bottom test tubes with screw cap containing 1ml of reagent - 50 Pasteur pipettes	50 kits per package 6 x 50 kits per box
0E003N	Copan SL Solution kit: - 50 12x80mm PET round-bottom test tubes with screw cap containing 1ml of reagent - 50 Pasteur pipettes	50 kits per package 6 x 50 kits per box
098CE.A	Copan SL Solution kit: - 20 12x80mm PP test tubes with screw cap containing 0.5ml of reagent - 20 sputum dippers	20 kits per package 10 x 20 kits per box
0U020N.A	Copan SL Solution kit: - 50 12x80mm PP test tubes with prehensile screw cap containing 1ml of reagent - 50 sputum dippers	50 kits per package 6 x 50 kits per box
097CE	Copan SL Solution in bulk: - 30 12x80mm PP tapered-bottom test tubes with screw cap containing 1ml of reagent	30 test tubes per package 10 x 30 test tubes per box
097CE.A	Copan SL Solution in bulk: - 30 12x80mm PP tapered-bottom test tubes with screw cap containing 1ml of reagent	30 test tubes per package 10 x 30 test tubes per box
099CE.A	Copan SL Solution in bulk: - 30 12x80mm PP tapered-bottom test tubes with screw cap containing 0.5ml of reagent	30 test tubes per package 10 x 30 test tubes per box
0E006N.A	Copan SL Solution in bulk: - 50 12x80mm PET round-bottom test tubes with screw cap containing 1ml of reagent	50 test tubes per package 6 x 50 test tubes per box
0E005N	Copan SL Solution in bulk: - 50 12x80mm PET round-bottom test tubes with screw cap containing 1ml of reagent	50 test tubes per package 6 x 50 test tubes per box
0E008N.A	Copan SL Solution in bulk: - 50 12x80mm PET round-bottom test tubes with screw cap containing 0.5ml of reagent	50 test tubes per package 6 x 50 test tubes per box
0E007N	Copan SL Solution in bulk: - 50 12x80mm PET round-bottom test tubes with screw cap containing 0.5ml of reagent	50 test tubes per package 6 x 50 test tubes per box
0U019N	Copan SL Solution in bulk: - 50 12x80mm PET round-bottom test tubes with prehensile screw cap containing 1ml of reagent	50 test tubes per package 6 x 50 test tubes per box

2U063S01	Copan Sputum Dipper in bulk: - 100 specimen transfer devices for prehensile cap	100 sputum dippers per package 10 x 100 sputum dippers per box
096CS01	Copan Sputum Dipper in bulk: - 100 specimen transfer devices	100 sputum dippers per package 10 x 100 sputum dippers per box

#### REAGENTS

The Copan SL Solution components are:

Components	Quantity (g/l)
Saline matrix	46.45
DTT (DL-dithiothreitol) <sup>1,2</sup>	2.0

#### REQUIRED MATERIALS BUT NOT PROVIDED

Specimen collection device; primary specimen transfer device (included and provided only for the kit versions); microorganism culture and isolation device.

#### STORAGE

This product is ready to use and requires no further preparation. The packed and never opened product, which has a clear/colourless appearance, can be stored at a temperature of 5-25 °C until the expiration date. Do not incubate or freeze. Keep away from light sources. Do not use after the expiration date. If incorrectly stored, its effectiveness will be compromised.

The codes 095CE, 095CE.A, 098CE.A, 097CE, 097CE.A and 099CE.A must immediately be used or within 24 hours from opening storing the product at 2-8 °C.

#### LIMITATIONS

- The conditions, times and volume of the specimen collected for culture are significant variables for obtaining reliable culture results. Follow the recommended specimen collection guidelines.
- Copan SL Solution is not suitable for recovery of *Mycobacterium spp.*
- The performance tests on Copan SL Solution were conducted using laboratory strains inoculated in the Copan SL Solution test tube and not human clinical specimens.
- If the specimen collected has a high viscosity, it may be necessary to vortex the test tube again in order to ensure complete fluidification. The vortex time should be decided by the operator during use by visually checking specimen fluidification.
- The sputum dipper, if provided, is not a quantitative specimen transfer device. The quantity transferred may vary depending on the manual skill of the operator and the nature of the specimen.

#### WARNINGS

- Single-use in vitro diagnostic device for professional use.
- Always use Copan SL Solution with filling proportional to the amount of specimen available in order to always ensure a 1:1 ratio.<sup>6,7,8</sup>
- Do not perforate the membrane in any way.
- Do not immerse the transfer device in the reagent prior to sampling.
- Do not aspirate the reagent from the test tube or pour it into the primary specimen.
- Do not syringe biological material or other reagents into the test tube.
- Always use a test tube rack for the non-self-supporting round-bottom version.
- Do not use Copan SL Solution if the reagent is not clear/colourless (e.g. pink or yellow).
- For the product codes 095CE, 095CE.A, 098CE.A, 097CE, 097CE.A, 099CE.A: after transferring the specimen, do not leave the transfer device or parts of it in the Copan SL Solution test tube.
- The use of this product in combination with diagnostic tests or diagnostic instrumentation should be validated by the end user before use.

#### INSTRUCTIONS FOR USE

- Take the correctly identified Copan SL Solution test tube and unscrew the cap.
- Transfer the specimen to the open test tube using a suitable transfer device, possibly provided in the kit version, respecting a ratio of about 1:1.<sup>6,7,8</sup>



NOTE I: Only if using the sputum dipper provided with the product codes 2U063S01 and 0U020N.A, **break** the transfer device in the test tube:

- Insert the sputum dipper in the test tube making sure that the tip touches the bottom.
- Break the sputum dipper with a decisive movement bending it 180° parallel to the test tube.
- Discard the upper broken off part of the sputum dipper.

NOTE II: Only if using the sputum dipper provided with the product codes 098CE.A and 096CS01, **do not break** the transfer device in the test tube:

- Insert the sputum dipper in the test tube making sure that the tip touches the bottom.
  - Turn the sputum dipper in the reagent 5-10 times to allow the solution to come into contact with the specimen.
  - Remove the sputum dipper from the test tube checking that the specimen has been transferred into it. Repeat the procedure if necessary.
  - Discard the sputum dipper.
- Screw the cap back onto the Copan SL Solution test tube taking care that the sputum dipper (if used) is recovered from the prehensile part of the cap.
  - Immediately vortex the test tube for 30 seconds at 2000-2500 rpm.

5. Leave the test tube at ambient temperature for at least 15 minutes and do not exceed a maximum time of 6 hours.
6. Homogenize the specimen by vortexing the test tube for at least 3 seconds at 2000-2500 rpm, checking that the specimen is actually suitable for seeding.
7. Process the specimen according to the required internal procedures.

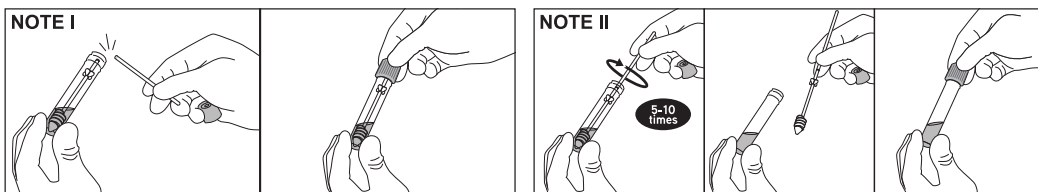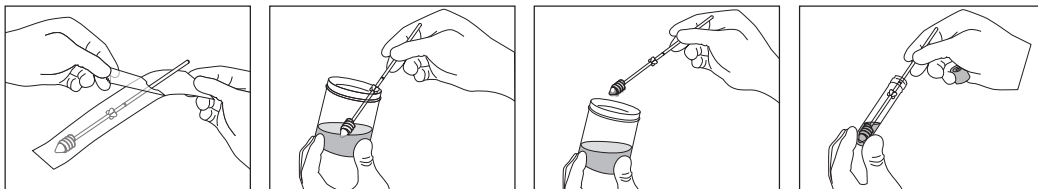**DISPOSAL**

Dispose of all waste according to the national laws. Use precautions for infected material when necessary.

**PERFORMANCE CHARACTERISTICS**

A test on the survival of the respiratory pathogens listed below stored in Copan SL Solution for up to 6 hours at ambient temperature has shown that the organisms remain vital:

STRAIN	CFU AT TIME ZERO	CFU AFTER 6 HOURS AT AMBIENT TEMPERATURE
<i>Pseudomonas aeruginosa</i> ATCC 27853	496	167
<i>Haemophilus influenzae</i> ATCC 10211	420	88
<i>Staphylococcus aureus</i> ATCC 6538	485	270
<i>Streptococcus pneumoniae</i> ATCC 6305	450	184
<i>Streptococcus pyogenes</i> ATCC 19615	466	178
<i>Moraxella catarrhalis</i> ATCC 25238	501	137
<i>Candida albicans</i> ATCC 10231	475	260
<i>Klebsiella pneumoniae</i> ATCC 700603	433	321

‡ The tests were conducted using ATCC strains rehydrated in the laboratory and not clinical specimens or artificial matrices.

**QUALITY CONTROL PROCEDURE**

- Starting from a fresh culture plate, prepare an inoculum equal to 0.5 McFarland of the organism under test in a test tube containing PBS.
- Prepare appropriate dilutions in PBS test tubes until obtaining a suspension with a concentration from 4x10<sup>2</sup> to 6x10<sup>2</sup> CFU/100 µl.
- Using a micropipette, inoculate 100 µl of the chosen dilution in the Copan SL Solution test tube.
- Vortex the inoculated test tube for 30 seconds at 2000-2500 rpm.
- Pipette 100 µl of inoculated Copan SL Solution onto a culture plate with an appropriate culture medium and with the aid of a spatula, evenly plate it on the entire surface of the culture medium.
- Incubate the plate at 35 °C ±2 °C for 24-48 hours in atmospheric conditions suitable for microorganism growth (specimen count at time zero of treatment).
- Leave the thus inoculated Copan SL Solution test tubes at ambient temperature (20-25 °C) for 6 hours.
- Vortex the inoculated test tube for 30 seconds at 2000-2500 rpm.
- Pipette 100 µl of inoculated Copan SL Solution onto a culture plate with an appropriate culture medium and with the aid of a spatula, evenly plate it on the entire surface of the culture medium.
- Incubate the plate at 35 °C ±2 °C for 24-48 hours in atmospheric conditions suitable for microorganism growth (specimen count at 6 hours of treatment).

**ACCEPTABILITY LIMITS:** reduction of the bacterial load not higher than 1 log<sub>10</sub> after 6 hours at ambient temperature with respect to the count obtained at time zero.

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#### BIBLIOGRAPHY

1. **A. Efthimiadis, M.M.M. Pizzichini, E. Pizzichini, J. Dolovich, F.E. Hargreave.** Induced sputum cell and fluid-phase indices of inflammation: comparison of treatment with dithiothreitol vs phosphate-buffered saline. *European Respiratory Journal* 1997; 10: 1336–1340.
2. **M R Hammerschlag, L Harding, A Macone, A L Smith and D A Goldmann** Bacteriology of sputum in cystic fibrosis: evaluation of dithiothreitol as a mucolytic agent. *J. Clin. Microbiol.* June 1980vol. 11 no. 6 552-557.
3. **J.Michael Miller and Shelley A. Miller,** Specimen Management in *Clinical Microbiology*, 3<sup>rd</sup> edition 2017, ASM Press, Washington DC, USA ; Section III- 103.
4. **Clinical Microbiology Procedures Handbook, Lynne S.Garcia, 3<sup>rd</sup> ed. Volume 1** , Section 2: Specimen, Collection, Transport and Acceptability T2.1-1; T2.1-2.
5. **Clinical Microbiology Procedures Handbook, Lynne S.Garcia, 3<sup>rd</sup> ed. Volume 1** , Section 3: Aerobic bacteriology, S3.11.2 lower Respiratory Tract Cultures .
6. **Methods of sputum processing for cell counts, immunocytochemistry and *in situ* hybridisation;** Q. Hamid, M.M. Kelly, M. Linden, R. Louis, M.M.M. Pizzichini, E. Pizzichini, C. Ronchi, F. Van Overveld, R. Djukanović *European Respiratory Journal* 2002 20: 19s-23s. **DOI:** 10.1183/09031936.02.00001902.
7. **Induced sputum analysis: step by step;** Zsoka Weiszhar, Ildiko Horvath *Breathe* 2013 9: 300-306; **DOI:** 10.1183/20734735.042912.
8. **Methodology for Sputum Induction and Laboratory Processing;** Guiot, J., Demarche, S., Henket, M., Paulus, V., Graff, S., Schleich, F., Corhay, J. L., Louis, R., Moermans, C. Methodology for Sputum Induction and Laboratory Processing. *J. Vis. Exp.* (130), e56612, doi:10.3791/56612 (2017).

THIS PAGE INTENTIONALLY LEFT BLANK

## INDEX OF SYMBOLS

Meaning / Significato / Signification / Bedeutung / Sens / Significado / Betyder / Betydning / Betyder / Anlami	Symbol / Simbolo / Símbolo / Symbole / Symbole / Símbolos / Symbol / Symbol / Symbol / Sembol
	Manufacturer / Fabricante / Fabricante / Hersteller / Fabricant / Fabricante / Producent / Produzenten / Tillverkare / Üretici
	In vitro diagnostic device / Dispositivo Diagnostico in Vitro / Dispositivo de diagnóstico in vitro / In-vitro-Diagnostikum / Dispositif de diagnostic in vitro / In vitro diagnostisk medicinsk anordning / Medicinsk anordning för in vitro-diagnostik
	CE marking / Marchio CE / Marca CE / CE-Kennzeichnung / Marquage CE / Marcação CE / CE-mærke / EU-merking / CE-märkning / CE işareti
	Do not reuse / Non riutilizzare / No reutilizar / Nicht zur Wiederverwendung / Ne pas réutiliser / Não voltare a usar / Brug det ikke igen / Må ikke gjenbrukes / Använd inte den igen / Tekrar kullannmayın
	Catalogue number / Numero di catalogo / Número de catálogo / Bestellnummer / Référence du catalogue / Referência do catálogo / Katalognummer / Katalognr / Katalognummer / Katalog numarası
	Temperature limitation / Limiti di temperatura / Límites de temperatura / Temperature Begrenzung / Limites de temperature / Limites de temperatura / Temperaturgrænser / Temperaturgrenser / Temperaturgrænser / Sicaklik limitleri
	Use by / Utilizzare entro / Fecha de caducidad / Verwendbar bis / Utiliser jusque / Prazo de validade / Anvendes før / Må brukes innen / Ska användas innan / Son kullanna tarihi
	Consult Instructions for Use / Consultare le istruzioni per l'uso / Consulte las instrucciones de uso / Gebrauchsanweisung beachten / Consulter les instructions d'utilisation / Consultar as instruções de utilização / Se vejledningen til brug / Se instruksjoner for bruk / Les bruksanvisningen / Kullanna talimatlarına bakınız
	Batch code (Lot) / Codice del lotto (partita) / Código de lote (Lote) / Chargencode (Chagenbezeichnung) / Code de lot (Lot) / Código do lote (Lote) / Seriennummer (parti) / Batch-nummer (parti) / Dra för att öppna / Lot (seri) kodu
	Contains sufficient for <n> tests / Contenuto sufficiente per <n> test / Contenido suficiente para <n> pruebas / Ausreichend für <n> Tests / Contenu suffisant pour <n> tests / Contémo suficiente para <n> testes / Indhold tilstrækkeligt til <n> prøver / Innhold tilstrekkelig for <n> test / Innehåller tillräckligt för <n> tester / <n> test için yeterli içerik
	Keep away from sunlight / Tenere lontano dalla luce solare / Mantener alejado de la luz solar / Von Sonnenlicht entfernt halten / Tenir à l'abri de la lumière du soleil / Manter afastado da luz solar / Holdes væk fra sollys / Må oppbevares langt unna direkte sollys / Får ej utsättas för solljus / Güneş ışığından uzak tutun
	Do not use if package is damaged / Non utilizzare in caso di confezionamento danneggiato / No utilizar en caso de paquete dañado / Bei Beschädigung der Verpackung nicht verwenden / Ne pas utiliser si l'emballage est abîmé / Não utilizar em caso de embalagens danificadas / Må ikke anvendes, hvis emballagen er beskadiget / Skal ikke brukes i tilfelle av skadet emballasje / Ska inte användas vid skadade förpackningar / Ambalajı hasar görmüşse kullannmayın

Copan Italia S.p.A.  
Via F. Perotti, 10  
25125 Brescia Italy  
Tel +39 030 2687211  
Fax +39 030 2687250

Email: [info@copangroup.com](mailto:info@copangroup.com)  
Website: [www.copangroup.com](http://www.copangroup.com)

North American Distributor:  
Copan Diagnostics Inc.  
26055 Jefferson Avenue  
Murrieta, CA 92562 USA  
Tel: 951-696-6957  
Fax: 951-600-1832

E-mail: [customerservice@copanusa.net](mailto:customerservice@copanusa.net)  
Website: [www.copanusa.com](http://www.copanusa.com)